**Qualitative Fit Test Kit**

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## EN: User Instructions: FT-10, FT-20, FT-30

### Contents:

- One Hood
- One Collar Assembly
- Nebulizer #1 (Sensitivity)
- Nebulizer #2 (Fit Test)
- Two Sets Replacement Nebulizer Inserts
- Sensitivity Solution (#1)
- Fit Test Solution (#2)
- 1 User Instruction

### INTENDED USE

The intended use of this product is to fit test particulate respirators or gas/vapor respirators with a particulate prefilter. Please refer to local regulations for suitability criteria.

**INGREDIENTS:** FT-10: Water, sodium saccharin; FT-30: Water, sodium chloride, denatonium benzoate

**CAUTION:** If there is evidence of solution contamination, immediately discard the bottle.

**CAUTION:** Denatonium benzoate is a very bitter chemical used to keep children from ingesting consumer products. Keep out of reach of children.

**NOTE:** If crystals are present, hold closed bottle under a warm stream of water or shake vigorously to dissolve the material.

### Regulatory Standard:

Follow all local laws and regulations for use of respiratory protection and fit testing. All regulations require that the person being fit tested is clean shaven. Do not fit test wearers who have facial hair that interferes with the seal surface of the respirator.

Note: Some regulations require that a medical evaluation be conducted prior to fit testing. Subjects should be informed of the ingredients of the fit test solutions and that they will be exposed to a fine mist.

### Preparation

1. Place hood on collar and position so drawstring is between flanges on collar. Attach hood to collar by tightening drawstring and tying with square knot or bow.
2. Prepare nebulizers: Make sure they are clean, with labels visible. Remove both plugs from holes.
3. Pour a small amount (approximately 3 ml) of the Sensitivity Test Solution (#1) into the nebulizer labeled "#1 Sensitivity Test Solution."
4. Pour the same amount of Fit Test Solution (#2) into the second nebulizer labeled "#2 Fit Test Solution."
5. Immediately recap the bottles.

## Sensitivity Test

This test is done to assure that the person being fit tested can detect the sweet or bitter taste of the test solution at very low levels. The Sensitivity Test Solution is a very dilute version of the Fit Test Solution.

The test subject should not eat, smoke, vape, drink (except water), or chew gum for 15 minutes before the test.

1. Have the test subject put on the hood and collar assembly without a respirator.
2. Position the hood assembly forward so there is about 6 in/15 cm between the subject's face and the hood window.
3. Instruct the test subject to breathe through his/her mouth with tongue extended.
4. Using Nebulizer #1 with the Sensitivity Test Solution (#1), inject the aerosol into the hood through the hole in the hood window. Inject ten squeezes of the bulb, fully collapsing and allowing the bulb to expand fully on each squeeze. Both plugs on the nebulizer must be removed from the openings during use. The nebulizer must be held in an upright position to ensure aerosol generation.
5. Ask the test subject if he/she can detect the sweet or bitter taste of the solution. If tasted, note the number of squeezes as 10 and proceed to the Fit Test.
6. If not tasted, inject an additional ten squeezes of the aerosol into the hood. Repeat with ten more squeezes if necessary. Note whether 20 or 30 squeezes produced a taste response.
7. Stop the test if the subject does not detect a sweet or bitter taste after 30 squeezes. Another type of fit test must be used.
8. Remove the test hood and give the subject a few minutes to clear the taste from his/her mouth. It may be helpful to have the subject rinse his/her mouth with water.

## Fit Test

1. Have the test subject don the respirator and perform a user seal check per the respirator manufacturer's user instructions.
2. Have subject wear any applicable safety equipment that may be worn during actual respirator use that could interfere with respirator fit (e.g. safety eyewear). Note: Some regulations require that a 5 minute comfort assessment is performed. Please see local regulations for more information.
3. Have the subject put on and position the test hood as before and breathe through his/her mouth with tongue extended.
4. Using Nebulizer #2 with Fit Test Solution (#2), inject the fit test aerosol using the same number of squeezes as required in the Sensitivity Test (10, 20, or 30). A minimum of ten squeezes is required, fully collapsing and allowing the bulb to expand fully on each squeeze. The nebulizer must be held in an upright position to ensure aerosol generation.
5. To maintain an adequate concentration of aerosol during this test, inject one-half the number of squeezes (5, 10, or 15) every 30 seconds for the duration of the fit test procedure (see Table 1). Periodically check the nebulizer to make sure that it is not clogged. If clogging is found, clean the nebulizer and retest. Table 1:

| Number of squeezes used in the Sensitivity test | Number of squeezes required at the start of the Fit Test | Number of subsequent squeezes every 30 seconds during the Fit Test |
|-------------------------------------------------|----------------------------------------------------------|--------------------------------------------------------------------|
| 1-10                                            | 10                                                       | 5                                                                  |
| 11-20                                           | 20                                                       | 10                                                                 |
| 21-30                                           | 30                                                       | 15                                                                 |

6. After the initial injection of aerosol, ask the test subject to perform the following test exercises for 60 seconds each:
  1. Normal breathing – In a normal standing position, without talking, the subject shall breathe normally.
  2. Deep breathing – In a normal standing position, the subject shall breathe slowly and deeply, taking caution so as not to hyperventilate.
  3. Turning head side to side – Standing in place, the subject shall slowly turn his/her head from side to side between the extreme positions on each side. The head shall be held at each extreme momentarily so the subject can inhale at each side.
  4. Moving head up and down – Standing in place, the subject shall slowly move his/her head up and down. The subject shall be instructed to inhale in the up position (i.e., when looking toward the ceiling).
  5. Talking – The subject shall talk out loud slowly and loud enough so as to be heard clearly by the test conductor. The subject can read from a prepared text such as the Rainbow Passage, count backward from 100, or recite a memorized poem or song.

**Rainbow Passage** When the sunlight strikes raindrops in the air, they act like a prism and form a rainbow. The rainbow is a division of white light into many beautiful colors. These take the shape of a long round arch, with its path high above, and its two ends apparently beyond the horizon. There is, according to legend, a boiling pot of gold at one end. People look, but no one ever finds it. When a man looks for something beyond reach, his friends say he is looking for the pot of gold at the end of the rainbow.

6. Bending over – The test subject shall bend at the waist as if he/she were to touch his/her toes. Jogging in place may be substituted for this exercise.
7. Normal breathing – Same as exercise a.
7. The test is terminated at any time the sweet or bitter taste of aerosol is detected by the subject because this indicates an inadequate fit. Wait 15 minutes and perform the sensitivity test again.
8. Repeat the fit test after redonning and readjusting the respirator. A second failure may indicate that a different size or model respirator is needed.
9. If the entire test is completed without the subject detecting the sweet or bitter taste of the aerosol, the test is successful and the respirator fit has been demonstrated.

At the end of each session or at least every four hours, discard the unused solution from the nebulizers. **DO NOT pour used solutions back into bottles.** Rinse the nebulizers with warm water to prevent clogging and shake dry. Wipe out the inside of the hood with a damp cloth or paper towel to remove any deposited Test Solution. 3M™ Respirator Cleaning Wipes may be used to clean non-disposable type respirator facepieces between fit tests.

Note: There is no published shelf life for the solutions. Any solution left in the nebulizers at the end of your fit test session should not be poured back into the bottles but rather should be discarded to avoid contamination of the solution remaining in the bottle.

## Replacement Parts

Part Number Description Packaging

FT-11 Sensitivity Solution (Sweet) 1 Each/Box; (55 ml Bottle) 6 Boxes/Case

FT-12 Fit Test Solution (Sweet) 1 Each/Box; (55 ml Bottle) 6 Boxes/Case

FT-13 Nebulizer 1 Each/Box; 3 Boxes/Case

FT-14 Test Hood 2 Each/Box; (2/Pack) 5 Boxes/Case

FT-15 Collar 1 Each/Box; 1 Box/Case

FT-31 Sensitivity Solution (Bitter) 1 Each/Box; (55 ml Bottle) 6 Boxes/Case

FT-32 Fit Test Solution (Bitter) 1 Each/Box; (55 ml Bottle) 6 Boxes/Case

## Important Notice

**Product Selection and Use:** Many factors beyond 3M's control and uniquely within user's knowledge and control can affect the use and performance of a 3M product in a particular application. As a result, customer is solely responsible for evaluating the product and determining whether it is appropriate and suitable for customer's application, including conducting a workplace hazard assessment and reviewing all applicable regulations and standards (e.g., OSHA, ANSI, etc.). Failure to properly evaluate, select, and use a 3M product in accordance with all applicable instructions and with appropriate safety equipment, or to meet all applicable safety regulations, may result in injury, sickness, death, and/or harm to property.

**Warranty, Limited Remedy, and Disclaimer:** Unless a different warranty is specifically stated on the applicable 3M product packaging or product literature (in which case such warranty governs), 3M warrants that each 3M product meets the applicable 3M product specification at the time 3M ships the product. 3M MAKES NO OTHER WARRANTIES OR CONDITIONS, EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OR CONDITION OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, OR ARISING OUT OF A COURSE OF DEALING, CUSTOM, OR USAGE OF TRADE. If a 3M product does not conform to this warranty, then the sole and exclusive remedy is, at 3M's option, replacement or repair of the 3M product or refund of the purchase price. Warranty claims must be made within one (1) year from the date of 3M's shipment.

**Limitation of Liability:** Except for the limited remedy stated above, and except to the extent prohibited by applicable law, 3M will not be liable for any loss or damage arising from or related to the 3M product, whether direct, indirect, special, incidental, or consequential (including, but not limited to, lost profits or business opportunity), regardless of the legal or equitable theory asserted, including, but not limited to, warranty, contract, negligence, or strict liability.

**Disclaimer:** 3M industrial and occupational products are intended, labeled, and packaged for sale to trained industrial and occupational customers for workplace use. Unless specifically stated otherwise on the applicable product packaging or literature, these products are not intended, labeled, or packaged for sale to or use by consumers (e.g., for home, personal, primary or secondary school, recreational/sporting, or other uses not described in the applicable product packaging or literature), and must be selected and used in compliance with applicable health and safety regulations and standards (e.g., U.S. OSHA, ANSI), as well as all product literature, user instructions, warnings, and limitations, and the user must take any action required under any recall, field action or other product use notice. Misuse of 3M industrial and occupational products may result in injury, sickness, or death. For help with product selection and use, consult your on-site safety professional, industrial hygienist, or other subject matter expert. For additional product information, visit [www.3M.com](http://www.3M.com).

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3M is a trademark of 3M. Used under
All rights reserved. Please recycle.